

# Clarity Urocheck 120 Urine Analyzer

## Step-by-Step Set-up Instructions

Read User's Manual for complete information.

1060 Holland Drive, Suite A  
Boca Raton, FL. USA 33487  
Toll Free: 877.722.6339  
Phone: 561.288.9028  
Support: 877.485.7877

Email: [techsupport@claritydiagnostics.com](mailto:techsupport@claritydiagnostics.com)  
Website: [www.claritydiagnostics.com](http://www.claritydiagnostics.com)

A. Plug AC adaptor into the back of the machine.



B. Plug power cord into power outlet.




C. Turn on Power Switch located on the back of the machine:



- Screen turns on and strip holder extends out and contracts back in (for calibration).



D. The Clarity Urocheck 120 Analyzer is preprogrammed and ready to use. Press start if you want to proceed with patient sample testing.

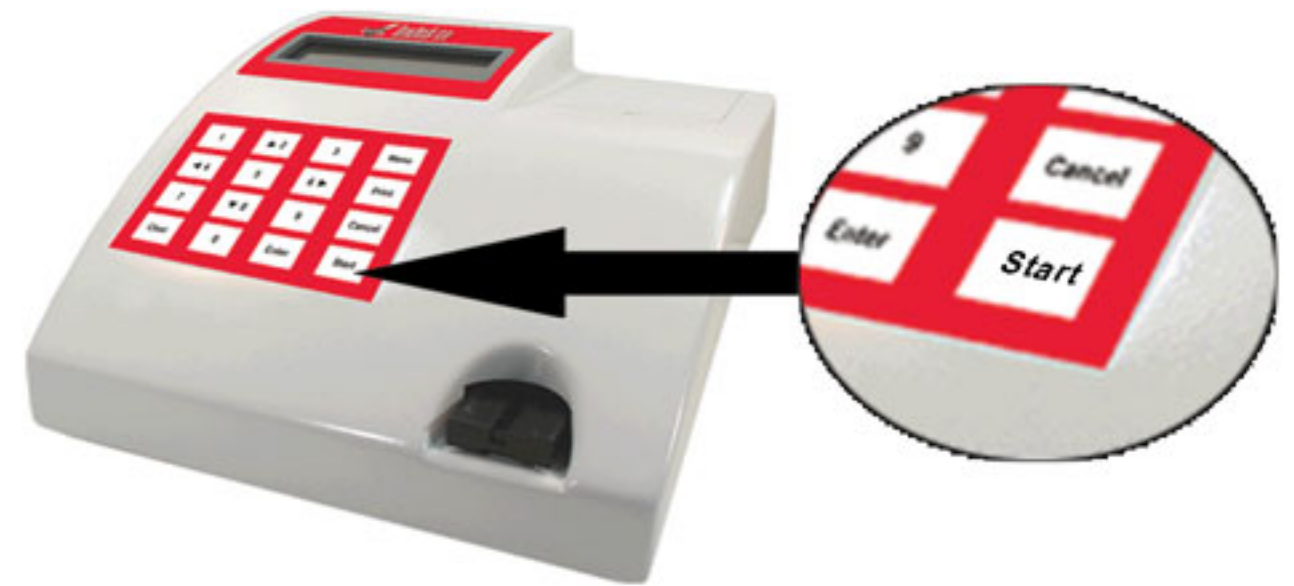

E. Obtain Urine Testing Kit (test strip bottle):

1. Remove bottle from box.
2. Remove cap.
3. Remove seal.



F. Prepare subject urine, control samples, and paper towel.



**IMPORTANT:** When finished testing make sure to return cap back onto test strip bottle and screw on tightly to insure product will last full usage cycle.



# Clarity Urocheck 120 Urine Analyzer

## Step-by-Step Set-up Instructions

Read User's Manual for complete information.

1060 Holland Drive, Suite A  
Boca Raton, FL. USA 33487  
Toll Free: 877.722.6339  
Phone: 561.288.9028  
Support: 877.485.7877


Email: techsupport@claritydiagnostics.com  
Website: www.claritydiagnostics.com

G. Now you are ready to perform the test:

1. Take one test strip and dip strip into patient sample, make sure to get each of the individual parameters wet.



2. Blot the urine strip on the paper towel to remove the excess urine.

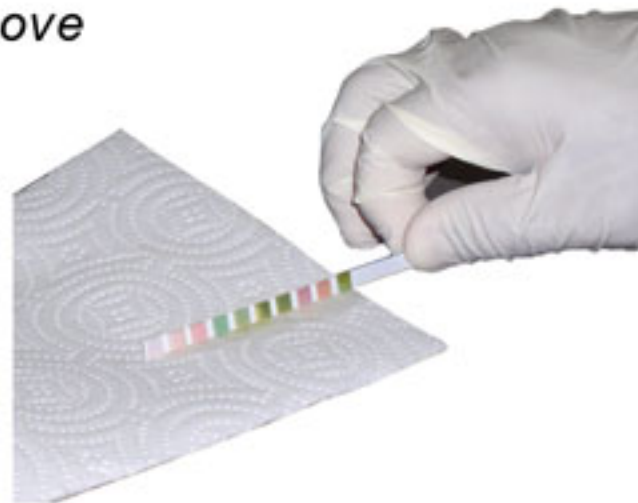

3. Place urine strip on stripholder.



4. Press START button to initiate test.



5. Test takes 65 seconds, the counter will count backwards from 65 to zero.



- Stripholder will contract and extend.

7. Results will print out of the analyzer when test is complete.



8. Take the used test strip off of the stripholder and throw it away.



9. Press CANCEL button twice to return to main menu.



**IMPORTANT:** Once done testing make sure to return cap back onto test strip bottle and screw on tightly to insure product will last full usage cycle.