




Drug Screening Cup Procedure Card

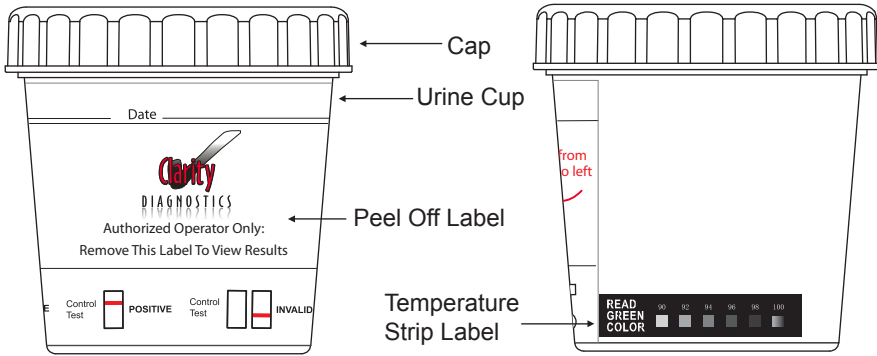

1 Remove the cap.

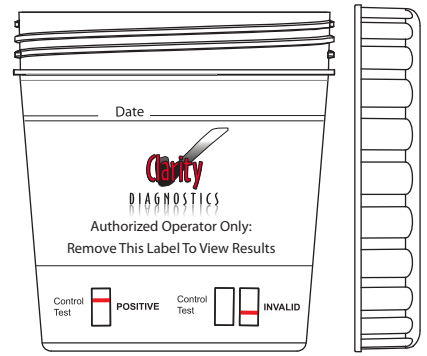





Donor provides specimen and replaces the cap.

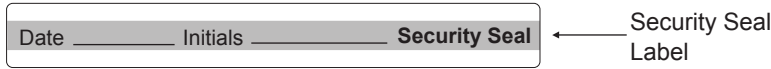

2 Read urine temperature between 2-4 minutes after voiding. Normal urine temperature should fall between the ranges of 90°F to 100°F.



3 Technician secures the cap tightly while the cup is on a flat surface, and dates and initials the security seal and places the security seal over the cap.

4 Technician peels off the label to view the results.

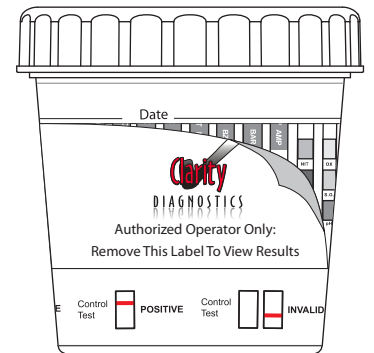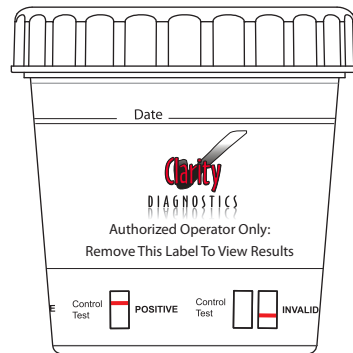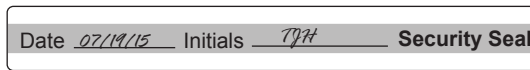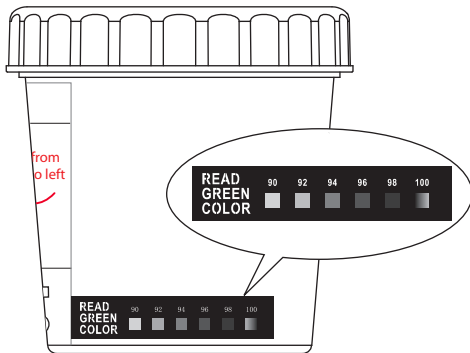

5 If adulteration test is included on the test cup, read the adulteration test results at 2 minutes, do not interpret the results after 5 minutes

6 Read the drug test results at 5 minutes, do not interpret the results after 10 minutes.

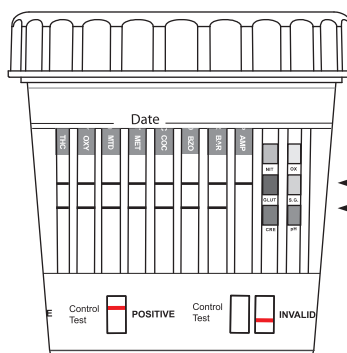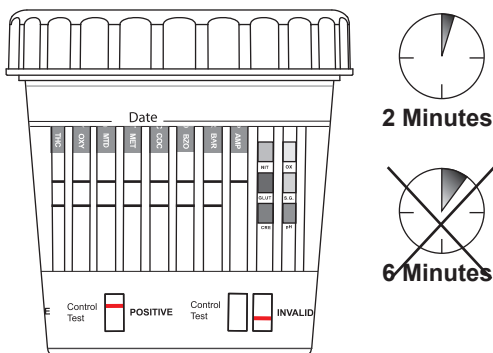

All positive results are presumptive and should be confirmed by an alternative method (e.g. GC/MS).

See the color chart and insert for interpretation.