



WELLlife™

COVID-19 / Influenza A&B Home Test

The WELLlife™ COVID-19 / Influenza A&B Home Test differentiates between SARS-COV-2, influenza A and influenza B antigens with a single test.

Features and Benefits

• Ease of use

Differentiate between SARS-COV-2, influenza A, and influenza B antigens with an easy nasal swab.

• Rapid & Convenient

Results are available in just 10 minutes, with kits that can be stored in 36°F-86°F(2°C-30°C), allowing testing anytime and anywhere.

• Reliable

The test demonstrates outstanding clinical performance, providing reliable results.

• Extended Detection Window

Offers a broader detection window for differentiation of SARS-COV-2, influenza A, and influenza B antigens within five (5) days of symptom onset.

• Pre-Loaded 12-Pack Retail Holder Design

Retail holders come pre-loaded with a total of 12 OTC 2ct packs for a total of 24 tests. Convenient for retail and pharmacy countertop displays or storage. Features a header card display with QR code for further product details and instructions.



Product Performance

	SARS-COV-2	Influenza A	Influenza B
Positive Percent Agreement (PPA)	87.5%	85.9%	86.8%
Negative Percent Agreement (NPA)	99.7%	99.7%	99.7%

In the USA, this product has not been FDA cleared or approved, but has been authorized by FDA under an Emergency Use Authorization. This product has been authorized only for the detection of proteins from SARS-CoV-2, influenza A, and influenza B, not for any other viruses or pathogens. The emergency use of this product is only authorized for the duration of the declaration that circumstances exist justifying the authorization of emergency use of in vitro diagnostics for detection and/ or diagnosis of COVID-19 under Section 564(b)(1) of the Federal Food, Drug, and Cosmetic Act, 21 U.S.C. §360bbb-3(b)(1), unless the declaration is terminated or authorization is revoked sooner.



Phone: 877-722-6339


sales@claritydiagnostics.com




WELLlife™

Quick 3-Step Test Protocol


- 1 Immerse the swab into the buffer tube and swirl the swab in the buffer. Ensure the sample is mixed thoroughly by making at least 10 circles.

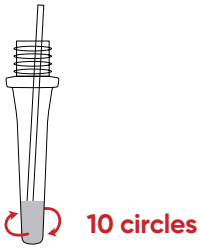

- 2 Invert the buffer tube and squeeze 4 drops of test sample into the sample well on the test cassette.

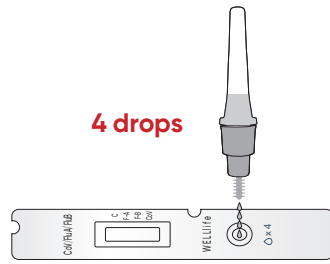


- 3 Start timer. Read results at 10 minutes.

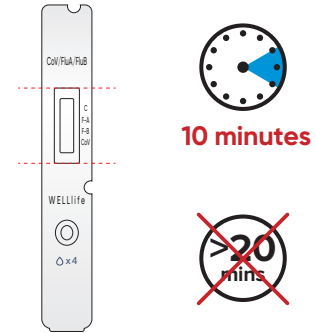


Clear Interpretation

Negative

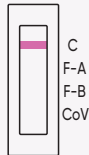


Positive

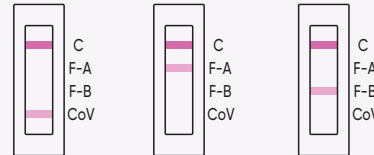

COVID-19 Positive Influenza A Positive Influenza B Positive

If the control line at "C" is visible and any other line or multiple lines at "F-A", "F-B" and/or "CoV" appear, the test is positive for that virus.

Invalid

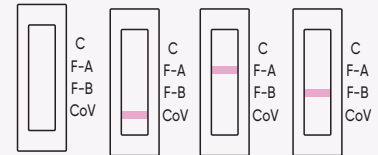

If a line is not visible at "C", even if any other line is visible in the results window, the result is considered invalid.

* Please refer to QRI for detailed results interpretation.

Ordering Information

Item Code	Description	Size
CLA-COVFLAB-OTC2	WELLlife™ COVID-19 / Influenza A&B Home Test	2 Tests / BX